

OraFID[®]

The OraFID[®] minitablet dispenser is a unique and innovative device that overcomes challenges of minitablet use by enabling easy and effortless dispensing of the exact number of minitablets desired.

The prefilled, single use multidose dispenser comes as a standard sized container. It is ready-to-use, so no filling or changing of lid is necessary.

As the dispenser is single use and cannot be refilled, the potential for contamination from any outside source is minimized.

The idea with minitablets is to allow for individualized dosing where each tablet

contains a low dose, and the patient's dose corresponds to a number of tablets.

By a simple manual movement, the right number of tablets are chosen and dispensed and the user can start the treatment with the correct number of minitablets.


- Patent protected dispenser
- Optimized for minitablets
- Counting accuracy



OraFID® Setup and Use

10 seconds of handling!

The dispenser comes ready to use and consists of a barrel and a bottom wheel. No filling of tablets required. The bottom wheel has a ring with numbers from 1–20, corresponding to the number of tablets that can be withdrawn at each single occasion. The amount of tablets should be set for each dispensing, hence the device always starts at the blue line.

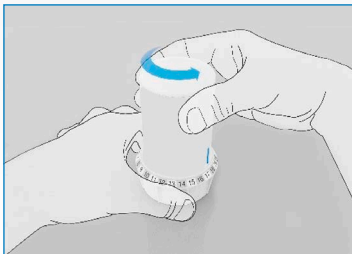


1.

To dispense medication, turn the barrel counter-clockwise to the wanted number of tablets. Use a steady, slow and firm movement. Do not back-up. If needed you can stop, take a new grip on the barrel and continue, but backing is not allowed.



2.

Once you have reached the wanted numbers, stop and turn the barrel clockwise back to, the stop symbol, where there is a firm stop. The correct number of minitabets automatically falls into the dispensing compartment.

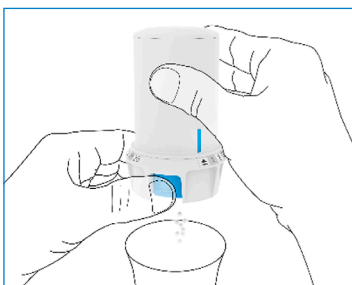

3.

Press the blue button on the bottom wheel to release the tablets.



orafid.com



Navamedic ASA, Henrik Ibsens gate 100, 0255 OSLO, Norway
info@navamedic.com